VISCOJECT™

Instructions for use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
Instrucciones de uso

SWISS MADE

Manufacturer:
MEDICEL AG
Luchten 1262
CH-9427 Wolfhalden
SWITZERLAND

Tel. +41 71 727 10 50
Fax +41 71 727 10 55
info@medicelag.com
www.medicelag.com

ENGLISH

Medicel VISCOJECT™
Single-use Lens Injection System
For injection of
ONE-PIECE FOLDABLE ACRYLIC LENSES

DESCRIPTION
The VISCOJECT™ injection system permits the implantation of foldable one-piece intraocular lenses through incisions of less than 2.5 millimetres for the very first time thanks to the new

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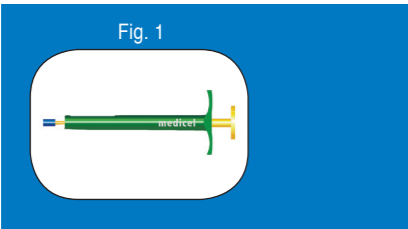
VISCOJECT™ hydraulics system. Not only the extremely thin special lenses conceived for micro-incision techniques can be injected using the VISCOJECT™ injection system, but also the majority of all foldable, one-piece acrylic lenses in any form with an optical diameter of 6.5 millimetres. The VISCOJECT™ injection system consists of the sterile VISCOJECT™ single-use INJECTOR with silicone cushion (Fig. 1) and the sterile VISCOGLIDE™ single-use cartridge (Fig. 2).

APPLICATION
Folding and injecting of one-piece foldable lenses into the capsular bag or sulcus after extra-capsular cataract extraction.

Three-piece foldable intraocular lenses are not suited for im-

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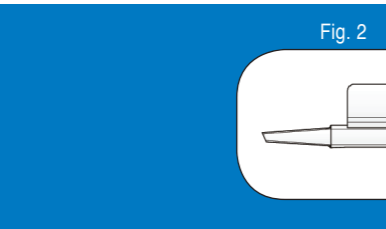
plantation using the VISCOJECT™ injection system.

INSTRUCTIONS FOR USE

1. Open the blister in the sterile area and remove the cartridge.
2. Fill the loading chamber of the VISCOGLIDE™ cartridge with viscoelastic solution (Fig. 3). Apply a small amount of the viscoelastic solution directly to the opening of the cartridge tunnel as well.

IMPORTANT: The lens should be injected immediately after being introduced into the cartridge tunnel. Viscoelastic materials may lose their lubricating properties when exposed to air for a longer period of time.

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3. Place the foldable lens on the central hinge of the opened cartridge as shown in Fig. 4. Align one-piece lenses in the direction of the cartridge regardless of the one-piece design.
4. Press the lens into both guide rails with rounded tweezers and gently move the lens back and forth horizontally in order to ensure that it can move freely (Fig. 5).
5. Ensure that the lens is correctly aligned and that the cartridge sides close. Make sure that neither the edge of the lens nor the haptic are wedged in.
6. Press the sides of the cartridge together until the click-lock mechanism engages (Fig. 6).

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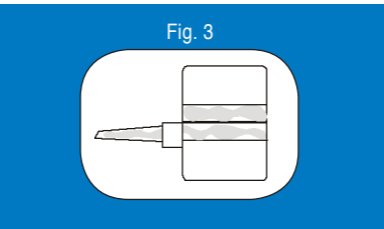
7. Insert the cartridge into the front end of the injector. Push the cartridge into the very front position (Fig. 7).
8. Carefully push the plunger forwards and ensure that the silicone cushion correctly enters the loading chamber. Continue to push the cushion until the inner spring begins to press together. Pull the plunger back a few millimetres and then push it forward again. This step ensures that the lens is correctly grasped.
9. Press the lens to the front part of the cartridge point outside of the eye. This prevents pressure generated by the VISCOJECT™ system from being transferred to the eye.

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10. Guide the point of the cartridge through the incision and push it across the iris to the near edge of the pupil.
11. Slowly press the plunger forward (the lens is pressed forward almost without contact as a result of the pressure created by VISCOJECT™). Depending on the lens model, support the correct ejection of the lens by slightly turning the injector. Slowly inject the lens into the eye and simultaneously withdraw the instrument from the eye.
12. Position the lens carefully and if necessary rotate it with the aid of the appropriate positioning hook.
13. Remove the viscoelastic material from the eye and the lens with the standard irrigation and aspiration techniques.

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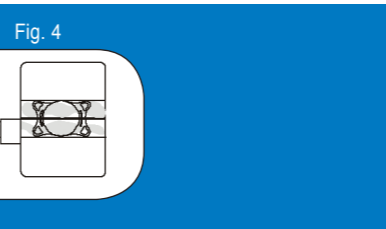
PLEASE ENSURE:
Do not sterilize / prepare cartridge and injector. Single use only.

OPERATIONAL PROCEDURE
The appropriate surgical techniques are the responsibility of the respective surgeon. He or she must assess the appropriateness of the relevant procedure based on his or her education and experience.

GUARANTEE AND LIABILITY LIMITATION
The manufacturer guarantees that this product has been manufactured with the appropriate care and assumes no liability for side effects or resulting damages, losses or costs that may arise as a result of the direct or indirect use of this product.

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